



## Data Management System (DMS) Pro User Guide



Eversense and the Eversense logo are trademarks of Senseonics, Incorporated. Other brands and their products are trademarks or registered trademarks of their respective holders.

Eversense DMS does not contain malware or computer viruses that might harm your computer. It is recommended you use an appropriate firewall program and an anti-virus program on your computer to prevent against malware and viruses.

To read the Eversense Privacy Policy, go to [global.eversenseddiabetes.com](https://global.eversenseddiabetes.com).

# Table of Contents

---

Glossary.....	3	4. DMS Dashboard.....	26
I. Introduction.....	4	Viewing Patient Eversense DMS Data	26
Intended Use	4	Setting Date Range	28
End User Licence Agreement and Privacy Policy	4	Eversense DMS Dashboard	29
Help and Support	4	5. Reports.....	33
2. Getting Started.....	5	Time in Target	35
Registering a clinic and creating your admin account	5	Time in Range	36
Accessing Eversense DMS Pro	7	Glucose Variability Report	38
Eversense DMS Pro Administrator	7	Glucose Trend Report	40
Navigation Menu	8	Glucose History Report	41
Eversense DMS Pro Clinic Staff User	13	Daily Glucose Report	42
Your Profile and the Navigation Menu	13	Glucose Distribution Report	44
Profile	14	Transmitter Log Report	45
Forgot Your Password	16	6. Settings.....	47
Home Page	17	Glucose	47
3. Permissions.....	18	7. Help.....	49
View	19	About	49
Add	21	8. Troubleshooting.....	50
Edit	24		
Admin	25		

# Glossary

**AutoSync** When this feature is turned on in the Eversense mobile app, your glucose data and settings will automatically get synced to the Eversense DMS every two hours.

**Blood Glucose** The glucose value obtained from a fingerstick test.

**Blood Glucose Meter** A commercially available device used to measure glucose using a blood sample from a fingerstick.

**Calibration** Blood glucose reading from a fingerstick sample entered in the Eversense App to check the accuracy of the system. With the Eversense System, there are two phases: Initialisation Phase during which 4 fingerstick tests are required, and the Daily Calibration Phase, during which 1 fingerstick test is required twice daily.

**CGM** Continuous Glucose Monitoring. Continuously monitoring your glucose levels from interstitial fluid every few minutes.

**Contraindication** A condition or circumstance in which a person should not use the device.

**EULA** End User Licence Agreement.

**Eversense App** Software program that is installed on a mobile device and displays CGM glucose data obtained from the smart transmitter.

**Eversense Client Application** A desktop application that allows Eversense users to upload the data on their smart transmitter to the Data Management System using a USB cable.

**Eversense Data Management System (DMS)** A web-based application compatible with the Eversense App where your glucose data is stored and can be viewed.

**Interstitial Fluid (ISF)** The fluid between cells in the body. The Eversense CGM measures glucose from an interstitial fluid sample, versus glucose in a blood sample obtained from a fingerstick.

**mg/dL** Milligrams per decilitre, a unit of measure that shows the concentration of a substance in a specific amount of fluid. In some countries, including the United States, glucose test results are reported as mg/dL, indicating how much glucose is in the blood when using a blood glucose meter, or how much glucose is in the interstitial fluid when using some CGM systems, like the Eversense CGM System.

**mmol/L** Millimoles per litre, a unit of measure that shows the concentration of a substance in a specific amount of fluid. In many countries, glucose test results are reported as mmol/L, indicating how much glucose is in the blood when using a blood glucose meter, or how much glucose is in the interstitial fluid when using some CGM systems, like the Eversense CGM System.

**Sensor Glucose** The glucose value obtained from the Eversense Sensor.

**Sensor** A device inserted subcutaneously for continually measuring interstitial fluid glucose levels.

**Smart Transmitter** A reusable device worn externally over the inserted sensor that powers the sensor and sends glucose information to the mobile device for display in the Eversense App.

# I. Introduction

---

## **Intended Use**

The Eversense Data Management System (DMS) Pro is a tool for healthcare professionals to manage and review glucose data of their patients that use the Eversense CGM System.

For a list of compatible devices and operating systems, please go to <https://global.eversenseddiabetes.com/compatibility>.

## **End User Licence Agreement and Privacy Policy**

Use of the Eversense DMS Pro is subject to the terms and conditions of the most current Eversense End User Licence Agreement and Eversense Privacy Policy. These documents are updated from time to time and are posted at [global.eversenseddiabetes.com](https://global.eversenseddiabetes.com).

## **Help and Support**

For questions regarding content in this DMS Pro User Guide, contact Customer Support toll free in the US at 844-SENSE4U (844-736-7348). Outside the US, call your local distributor or visit [global.eversenseddiabetes.com](https://global.eversenseddiabetes.com) to locate your local distributor.


## 2. Getting Started

### Registering a clinic and creating your admin account

To register your clinic, you must also register a clinic admin account. To do so, go to **globalpro.eversensedms.com**. After registering, your clinic will be assigned a Clinic Number which will be used as your identifier for patients to be able to request to join your clinic.

#### To register your clinic and create your admin account:

1. Go to **globalpro.eversensedms.com**
2. Click on “**Sign Up**” on the login screen

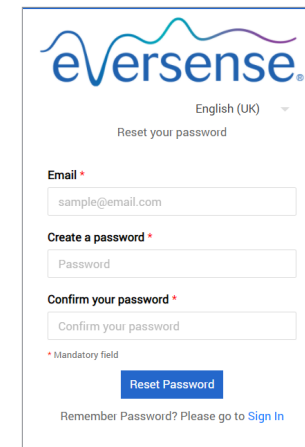


The screenshot shows the Eversense registration page. At the top is the Eversense logo with a blue and purple wave graphic. Below the logo is a language dropdown menu set to 'English (UK)' and a link to 'Reset your password'. The form contains three mandatory fields, each marked with a red asterisk: 'Email' (containing 'sample@email.com'), 'Create a password' (containing 'Password'), and 'Confirm your password' (containing 'Confirm your password'). A legend indicates that the red asterisk denotes a 'Mandatory field'. At the bottom of the form is a blue 'Reset Password' button and a link that says 'Remember Password? Please go to Sign In'.

### 3. Fill out registration form

**IMPORTANT:** You cannot use the same email address as a username for Eversense DMS Pro and a personal Eversense DMS account.

### 4. Click “Submit”

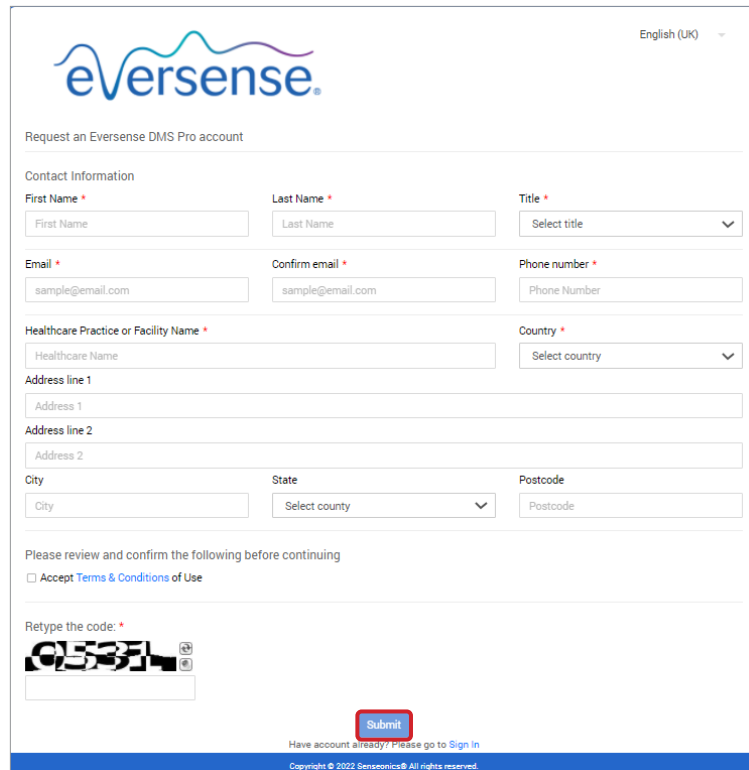

Request an Eversense DMS Pro account

English (UK)

Contact Information

First Name \* Last Name \* Title \*

Email \* Confirm email \* Phone number \*

Healthcare Practice or Facility Name \* Country \*

Address line 1

Address line 2

City State Postcode

Please review and confirm the following before continuing

☐ Accept [Terms & Conditions of Use](#)

Retype the code: \*

Submit

Have account already? Please go to [Sign in](#)


Copyright © 2022 Senseonics® All rights reserved.

### 5. You will be emailed a link to activate your account when your clinic and admin accounts are approved

The link will expire after 72 hours.

### 6. Once you click the link to activate your account, you will be prompted to create your Eversense DMS Pro password

After your password is created, you can log in and see your Clinic Number at the top of the screen next to your clinic name.

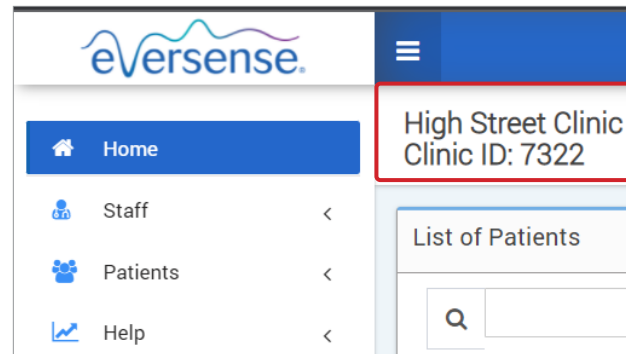



## Accessing Eversense DMS Pro

To access Eversense DMS Pro, go to [globalpro.eversensedms.com](https://globalpro.eversensedms.com). You will be directed to the login page after selecting your region.

### Eversense DMS Pro Administrator


Eversense DMS Pro requires at least one account administrator who can:

- Register the clinic, which generates the unique clinic ID.
- Invite other staff members to join the clinic site and set up their permissions.
- Change staff member permissions.
- Delete staff members from the clinic admin account.
- Assist in password reset for other staff members.

### DMS Pro Permission Levels

In Eversense DMS Pro, there are different permissions a clinic staff user can have. The permissions are **Admin**, **View**, **Edit**, and **Add**. The admin of your clinic assigns the permissions to the other clinic staff users.

- **Admin:** Can View, Edit, and Add.
- **View:** Allows the clinic staff user to view the accepted patients list and view the data in each patient's DMS account.
- **Add:** Will allow the clinic staff user the same permissions as View, plus the ability to accept patient requests to join your clinic, and to send patients invitations to join your clinic.
- **Edit:** Allows the clinic staff user to edit a patient's information in DMS Pro, unsubscribe a patient from the clinic, and view their Eversense DMS data. Any changes made in DMS Pro will not change any information the patient has saved in their personal account.

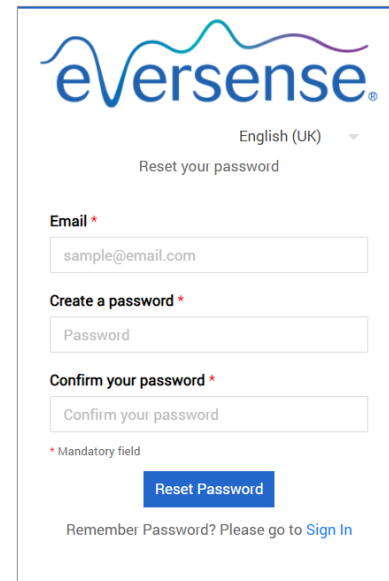


## Navigation Menu

At the top left-hand side of the Home page is the navigation menu. You can open and close the navigation menu by clicking . Click any button on the menu to navigate to that selection.


### Note:

- You can also select your preferred language by clicking the flag icon in the top right corner of the page.
- Your navigation menu may look different depending on the permission assigned to you by your clinic admin.


#### Admin

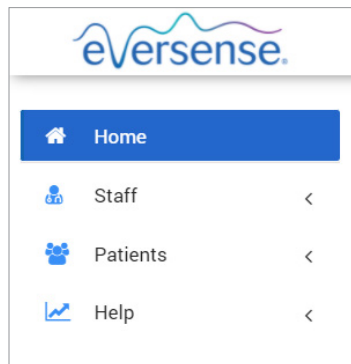


#### View

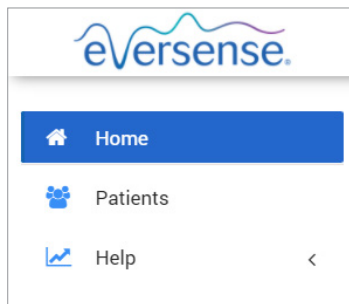

#### Add

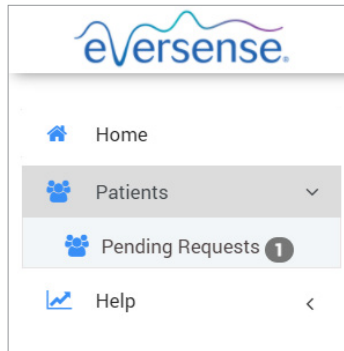


#### Edit

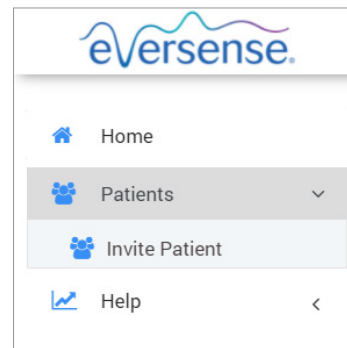


### To add staff members:

1. Click “**Staff**” on the left navigation bar
2. Click “**Add Staff**”
3. Fill out their contact information, and select their title
4. Review and edit the permissions as needed  
Multiple permissions may be assigned.
5. Click “**Send**”
6. The clinic staff will receive the invitation via email, and they must click on the verification link in the email to log in using the temporary password provided in the email.  
After successfully logging in, they will be prompted to create a new password.

**IMPORTANT:** Default permissions are assigned based on the title selected. If the title of a user is changed, you should check the permissions and edit as needed.

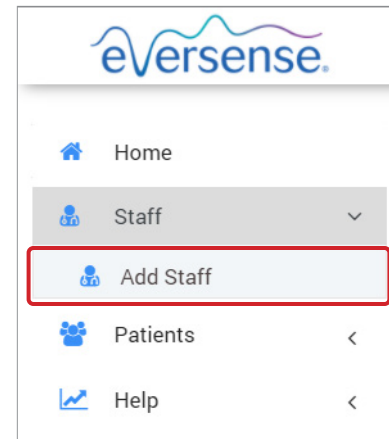




### To edit staff member information and permissions:

1. Click “Staff” on the left navigation bar
2. Click  to open their information page
3. From this page you can:
  - a. Edit their name and/or title
  - b. Update their permissions
4. Click “Update” when finished

**IMPORTANT:** If the title of a user is changed, you should check the permissions and edit as needed.

List of Patients

☐ Show deactivated patients

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
<a href="#">Martin</a>	Paul		Male	Jun 01, 1967	Laura Yankov	Active	✓		

Edit Clinic Patient

✕

Email \*

jeangarc97@gmail.com

MRN

Medical Record Number

Provider

Jon Johnson x

Last Name \*


Garcia

First Name \*

Jean

Date Of Birth \*


10/07/2000



Cancel

Update

## To deactivate clinic staff accounts:

1. Click “Staff” on the left navigation bar
2. Click  to open their information page
3. Choose Inactive on the bottom of the page

When an account is inactive, the clinic staff will no longer be able to add, edit, or view any patient accounts in DMS Pro.

Edit Clinic Patient

Email \*

jeangarc97@gmail.com

MRN

Medical Record Number

Last Name \*

Garcia

Date Of Birth \*

10/07/2000

Provider

Jon Johnson x

First Name \*

Jean

Cancel

Update

**To assist in password reset for clinic staff:**

1. Click **“Staff”** on the left navigation bar


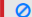
2. Click 

Find the clinic staff member and click .

3. Enter the new password and confirm it

4. Click notify user if you wish to notify the staff member via email that their password has been reset

5. Click **“Reset Password”**

List of Patients									
<input type="text"/>		<input type="checkbox"/> Show deactivated patients		Patients by Provider		Export Data			
Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
Martin	Paul		Male	Jun 01, 1967	Laura Yankov	Active	✓		

Reset clinic staff password ×

Username

Jon Johnson

New password

New password

Confirm Password

Confirm Password

☐ Notify user

Reset Password

Cancel

## Eversense DMS Pro Clinic Staff User

Your admin will create an account for you in Eversense DMS Pro. You will receive an email with a temporary password and a link to activate your account and to create a permanent password. Click on the link to complete your account set up, and display the Home page. Once your account is activated, you can go to **globalpro.eversensedms.com** to log in at any time.

### Your Profile and the Navigation Menu

Upon successful sign in, the Home page will be displayed.

- On the top right of the page you can access your **Profile** information.
- On the top left side of the page you will see a **Navigation Menu**. Your navigation menu may look different depending on the permission assigned to you by your clinic admin.

Best diabetic Care  
Clinic ID: 7323

List of Patients

Search:   ☐ Show deactivated patients

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share	Unsubscribe
No Data to display								

## Profile

Located at the top right of the Home page, this feature allows you to change your name or password, edit your user profile, or sign out of the Eversense DMS Pro.



### Change your password:

#### 1. Click your username

A drop-down will appear displaying your profile picture, your most recent log in, and options to view/edit your profile or sign out.

#### 2. Click “Profile” to access User Profile Settings


#### 3. Click “Change Password” to display and change your current password

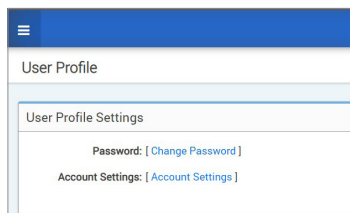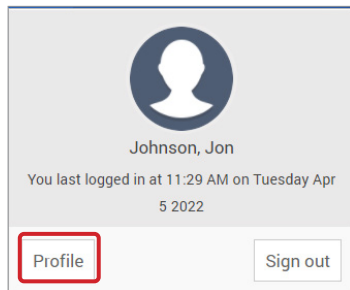

#### 4. Enter your Current Password

#### 5. Enter and confirm your new password

#### 6. Click “Change Password” to save your changes

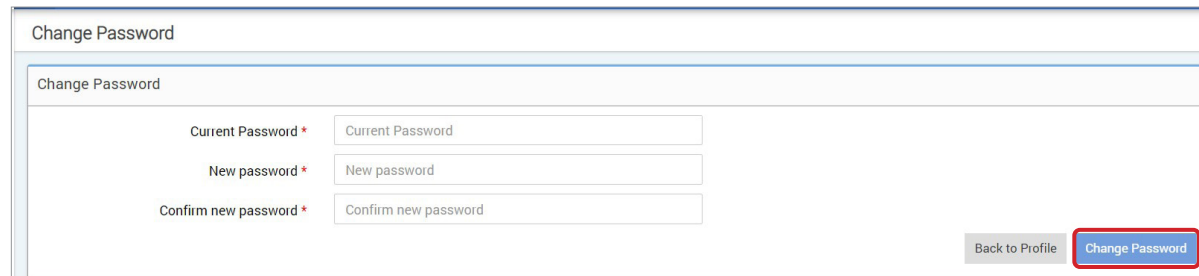A screenshot of the "Change Password" form. The form has a title "Change Password" at the top. Below the title, there are three input fields: "Current Password \*", "New password \*", and "Confirm new password \*". Each field has a placeholder text: "Current Password", "New password", and "Confirm new password" respectively. At the bottom right of the form, there are two buttons: "Back to Profile" and "Change Password" (highlighted with a red box).




## To adjust your account information:

1. Click “Account Settings”
2. Enter the information you will be changing

You can change your profile picture by pressing the **Choose File** button, and choosing a picture from your computer’s files.

3. Click “Save” to save your changes


The User Profile Settings page will appear again. To return to the Home page, click “**Home**” at the top left on the navigation panel. Patients using Eversense DMS or clinic staff users using Eversense DMS Pro will not be able to view your profile picture.

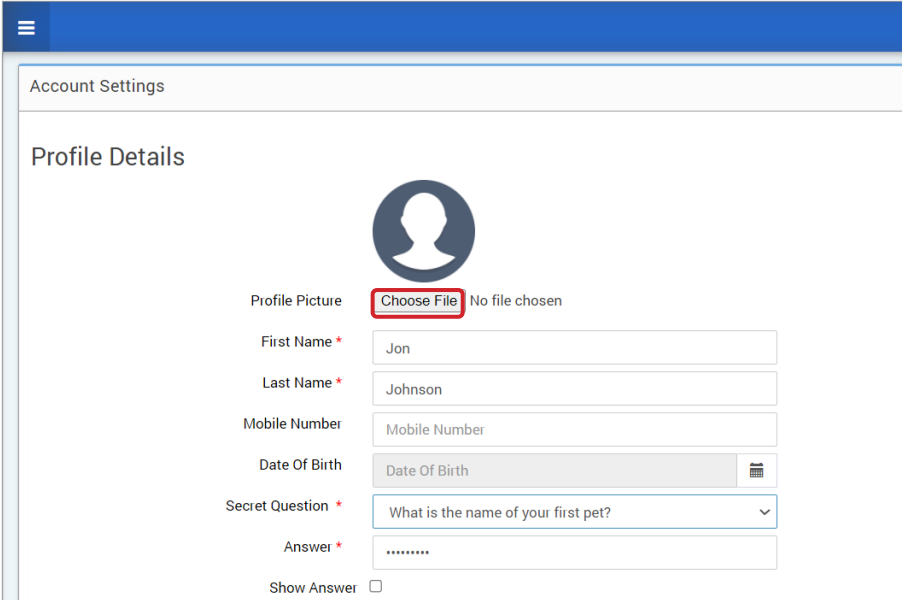

The screenshot shows the 'Account Settings' page with the 'Profile Details' section. The 'Profile Picture' field has a 'Choose File' button highlighted with a red box, and the text 'No file chosen' next to it. Below this are input fields for 'First Name' (Jon), 'Last Name' (Johnson), 'Mobile Number' (Mobile Number), and 'Date Of Birth' (Date Of Birth). There is also a 'Secret Question' dropdown menu set to 'What is the name of your first pet?' and an 'Answer' field with asterisks. A 'Show Answer' checkbox is at the bottom.

## Forgot Your Password

1. Click **“Forgot your password”** on the Sign In page
2. Enter your email when prompted
3. Click **“Send Mail”**

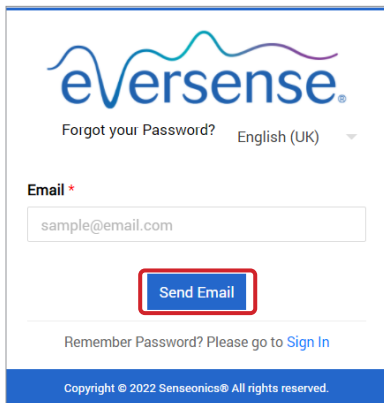

eversense®

Forgot your Password? English (UK) ▾


Email \*

sample@email.com

Send Email

Remember Password? Please go to [Sign In](#)

Copyright © 2022 Senseonics® All rights reserved.

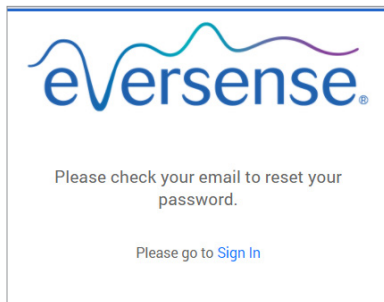

eversense®

Please check your email to reset your password.


Please go to [Sign In](#)

*A message will display to let you know an email has been sent to you with instructions for resetting your password.*

4. Open the email and click the website link

You will be directed to a web page to create and confirm a new password.

5. Click **“Reset Password”**

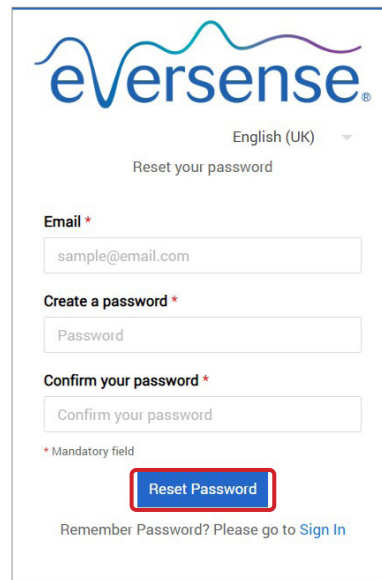

eversense®

English (UK) ▾

Reset your password

Email \*

sample@email.com

Create a password \*

Password


Confirm your password \*

Confirm your password

\* Mandatory field

Reset Password

Remember Password? Please go to [Sign In](#)

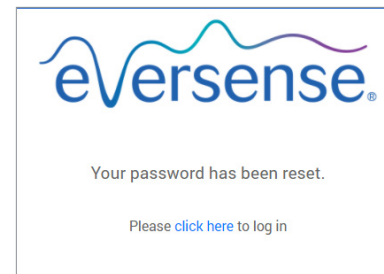

eversense®

Your password has been reset.

Please [click here](#) to log in

*A message will appear, confirming your password has been reset. Select the **“click here”** link to return to the Sign In page.*

# Home Page

The Home page displays a list of active patients based on the filters you select, and system defaults. Until patients have been added to your DMS Pro clinic, that page will remain blank.

English (UK)

james.anderson@mailinator.com

Trust Care Center  
Clinic ID: 7330

List of Patients

Q

☐ Show deactivated patients

Patients by Provider

Export Data

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
No Data to display									

# 3. Permissions

Your DMS Pro admin will assign you a permission.

3

In Eversense DMS Pro, there are different permissions a clinic staff user can have within a clinic. The permissions are **Admin**, **View**, **Edit**, and **Add**. The DMS Pro admin of your clinic assigns the permissions to you. All permission levels are able to search, filter, and export patient lists.

## Search, Filter, Export Patient List


List of Patients

☐ Show deactivated patients

Patients by Provider

Export Data

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
Garcia	Jean		Female	Oct 07, 2000	Jon Johnson	Active	✓		

Click		Patients by Provider	Export Data
To	Search	Filter	Export
How	You can search for patients by name in the search bar above the List of Patients.	You can also filter the List of patients by their healthcare provider by clicking Patients by Provider. <ol style="list-style-type: none"><li>1. Click “Patients by Provider”</li><li>2. Check the box of the provider whose patients you would like to view</li><li>3. Click “Proceed”</li></ol>	You can export the patient list as an Excel sheet when you click Export Data.

# View

To view a patient's data:

- 1. Click on the patient's name

List of Patients

The record has been updated successfully

Q

☐ Show deactivated patients

Patients by Provider


Export Data

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
Garcia	Jean		Female	Oct 07, 2000	Jon Johnson	Active	✓		

## 2. The patient's DMS information will be displayed

For more information, see *Viewing Patient Eversense DMS Data*.

3

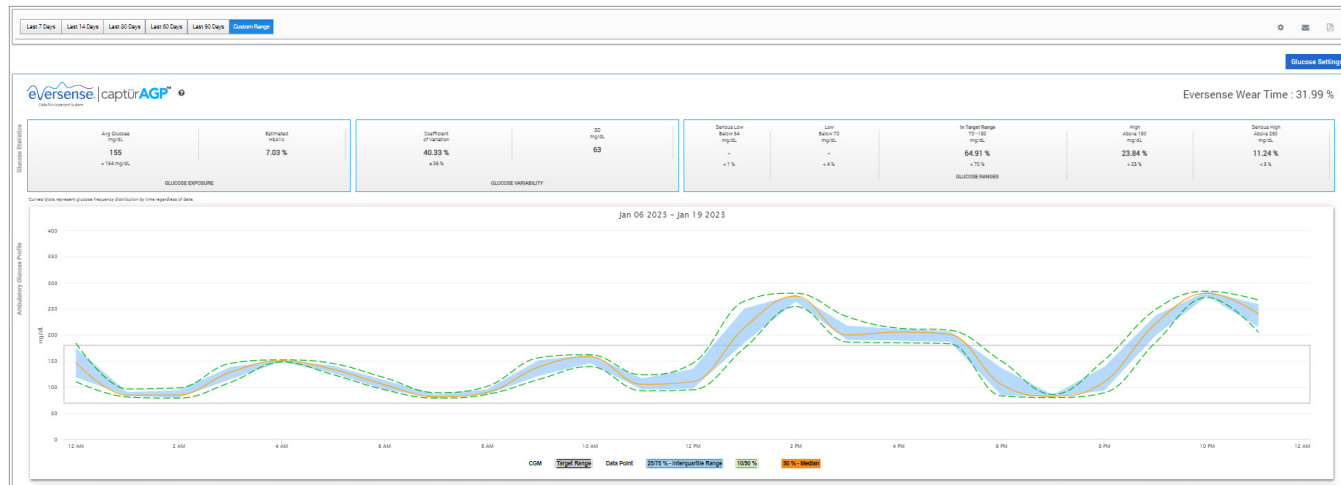


## Add

To add an Eversense CGM user to your patient list, you can either send them an invitation through DMS Pro, or they can request to join your clinic.


The patient must already have an Eversense DMS account. Only Eversense CGM users can be added to your Eversense DMS Pro patient list. For more information on how a patient can create a personal Eversense DMS account, they can visit [global.eversenseddiabetes.com](https://global.eversenseddiabetes.com) and view the Eversense DMS User Guide.

Once the patient has created a personal account, you can send them an invitation, or they can send a request to join your clinic.

### To send a patient invitation:

1. Click Patients 
2. Click Invite Patients 
3. Type in the email address that the user has used to register their Eversense account


**Note:** An invitation cannot be sent if the email has not been registered with Eversense DMS.

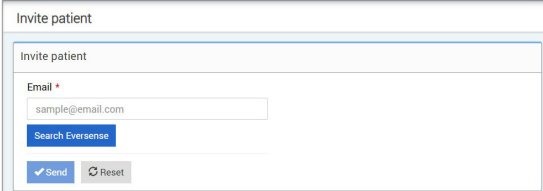

The screenshot shows a web interface titled "Invite patient". It contains a form with a label "Email \*" and a text input field containing "sample@email.com". Below the input field is a blue button labeled "Search Eversense". At the bottom of the form are two buttons: a blue button with a checkmark and the word "Send", and a grey button with a circular arrow and the word "Reset".




4. Click “Search Eversense” to confirm if that email address has been registered with Eversense
5. If the email has been registered, you will be prompted to enter patient information
  - a. To enter a MRN (optional)
  - b. To select a provider within your clinic for this patient
  - c. Edit their first or last name
  - d. Enter their date of birth (optional)
  - e. Enter their gender (optional)
6. Once complete, click “Send”

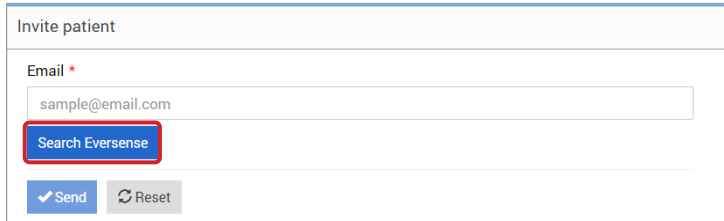An email invitation will be sent to the patient.




This is a larger, clearer screenshot of the "Invite patient" form. It shows the "Email \*" label, the input field with "sample@email.com", and the "Search Eversense" button which is highlighted with a red rectangular border. Below the input field are the "Send" and "Reset" buttons.

3







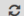



7. You will be asked if you want to send another invitation. If you click No, you will be returned to the patient list screen, where you will see the patient you just invited
8. Until the patient has accepted the invitation, there will be a  in the data share column
9. Once they accept the invitation, the  will turn into a , and you will now be able to click on their name to view their Eversense DMS data

List of Patients

The record has been updated successfully


 ☐ Show deactivated patients

[Patients by Provider](#) [Export Data](#)

Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	Data Share		Unsubscribe
<a href="#">Garcia</a>	Jean		Female	Oct 07, 2000	Jon Johnson	Active			
<a href="#">Martin</a>	Paul		Male	Jun 01, 1967	Laura Yankov	Active			
Muller	Ingrid		Female	Dec 01, 1980	Jon Johnson	Active			



## Accepting a patient request


A patient can send a request to join your clinic from their personal Eversense DMS account if you have provided them with your Eversense DMS Pro clinic number. They can enter your clinic number to send you a request in the Data Share page of their DMS account. To view a list of pending requests, click .

### To accept a patient request:

#### 1. Click Patients


#### 2. Click Pending Requests

You will see a number next to Pending Requests. This number shows you how many requests are currently in the list. If there is no number, then there are no pending requests.

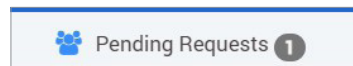

List of Pending Patients							
<input type="text" value="Search Pending Patient"/>		<button>Export Data</button>					
Patient last name	Patient first name	MRN	Gender	DOB	Provider Name	Status	
Wood	Shawn		Male	Apr 01, 2004	Jon Johnson	Pending	<button>Accept</button> <button>Decline</button>
1 Total							<div>&lt;&lt; &lt; 1 &gt; &gt;&gt;</div>

#### 3. Click “Accept” to add them to your clinic

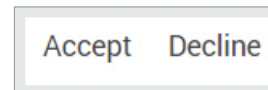

- When you click Accept, you will be prompted to assign them to a provider from your clinic.
- If you click Decline, their request will be removed, and they will not be added to the clinic list.

The Eversense CGM user will be notified via email when you accept or reject their request.

## Edit

The **Edit** permission allows you to view and edit a patient's information in DMS Pro, or unsubscribe a patient from the Eversense DMS Pro clinic list. Any changes made in DMS Pro will not change any information the patient has saved in their personal account.

3


### To edit a patient's information:

1. Click Patients 

2. Click 

3. This will open up a window with their information

You can edit all of their information, except for their email address.

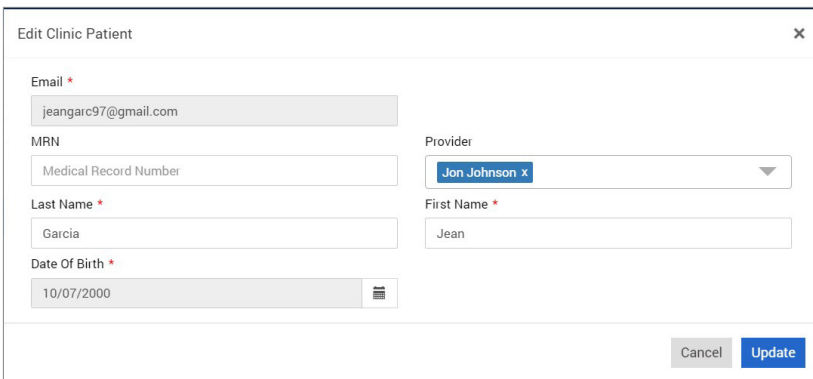

### To unsubscribe a patient from the clinic:

1. Click Patients 

2. Click 

3. Click “Yes”

The patient will now be removed from the DMS Pro clinic list and their Eversense data will no longer be available to view.

With the Edit permission, you are also able to accept Pending Requests from patients who want to join the clinic. See *To accept a patient request* for more information.

## Admin


The **Admin** permission allows you to have all of the same permissions as **View**, **Add**, and **Edit**, plus you are able to add, edit, and remove staff members. To learn more about the **Admin** permissions, See *Eversense DMS Pro Administrator*.

## 4. DMS Dashboard

### Viewing Patient Eversense DMS Data

When a patient is added to your clinic, you can click on their name to open a Home page that displays their Eversense DMS data. This page is almost identical to what the patient sees when they log in to their personal Eversense DMS account.

4

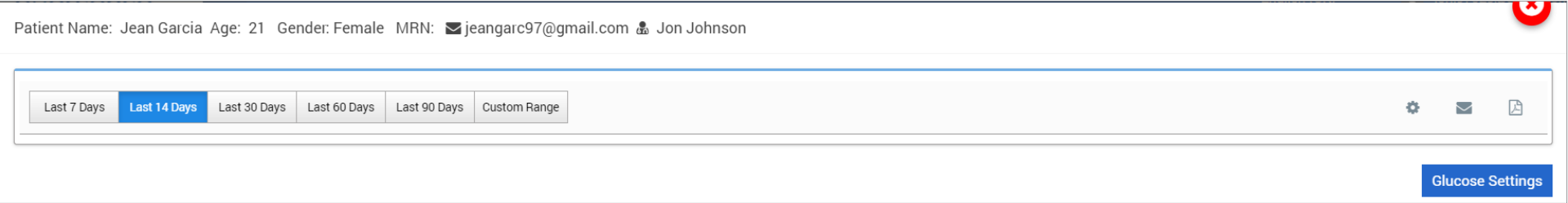

The screenshot shows a web interface for viewing patient data. At the top, it displays patient information: "Patient Name: Jean Garcia Age: 21 Gender: Female MRN: jeangarc97@gmail.com Jon Johnson". Below this is a horizontal bar with date range filters: "Last 7 Days", "Last 14 Days" (which is highlighted in blue), "Last 30 Days", "Last 60 Days", "Last 90 Days", and "Custom Range". To the right of these filters are three icons: a gear for settings, an envelope for email, and a document for printing. In the bottom right corner of the interface is a blue button labeled "Glucose Settings". A red close button with a white 'X' is located in the top right corner of the window frame.

**IMPORTANT: Any changes made to the settings or date ranges in DMS Pro WILL NOT affect the settings of the patients, Eversense DMS account or Eversense mobile app.**

At the top of the Home page their name, age, gender, MRN, email and assigned health-care provider is displayed.

You can exit this page at any time by clicking the  in the top right corner of the window or Close at the bottom right of the window.

**The Home page contains several sections:**

- Date Range Selector
- Eversense Wear Time
- Ambulatory Glucose Profile (AGP)
- Reports
- System Information
- Short cuts to print or email

Until the patient has uploaded sensor glucose data from the Eversense mobile app or their smart transmitter, no data will be displayed.

**IMPORTANT: If the patient has autosync turned on, their data will be automatically uploaded to DMS from their mobile app. They can also manually upload their glucose data from their smart transmitter via a USB cable. For more information on data upload, visit [eversenseddiabetes.com](https://eversenseddiabetes.com) to view the Eversense DMS User Guide.**

## Setting Date Range

At the top of the Home page, you can view reports based on a predefined date range, or a custom date range you set.

### To adjust the date range settings:



1. If you would like to view a set of data that is not within one of the predefined date ranges, click the button labelled “Custom Range” to display a drop-down calendar.



*This will display a pop-up that allows you to select your Custom Date Range.*

**Note:** Data accessible through DMS Pro goes as far back as one year

2. Select the desired report *Start Date*
3. Select the desired report *End Date*
4. Click “Ok” when complete

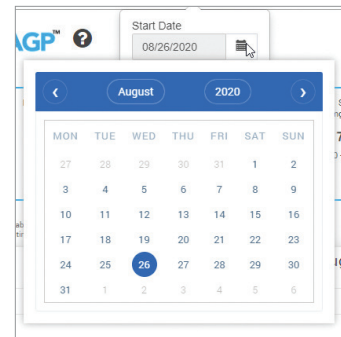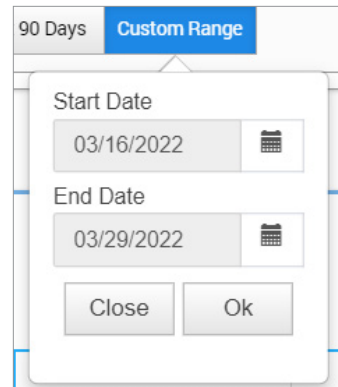


# Eversense DMS Dashboard

The Eversense DMS Dashboard is the default page that your patient will see each time the patient log into DMS. This page contains the patient’s Eversense Wear Time, Captur AGP reports, and the patient’s System Information.

## Eversense Wear Time

Eversense Wear Time is the percentage of time the smart transmitter was worn, over the date range the patient selected. 100% is 23.5 hours of daily smart transmitter wear time. The percentage does not include times in which the glucose is not displayed. For example: the beginning of initialisation phase or when an alert is activated that prevents glucose from being displayed.

 **Eversense Wear Time**

61.2 %

**System Information:** Located at the top of the AGP Dashboard is the System Information section.


*This section shows the patient’s:*

- Transmitter Serial Number
- Sensor Serial Number
- Glucose settings: Glucose settings can be changed in the DMS as well as through the Eversense mobile app (see *Settings* for more information).

System Information	Glucose Settings		
Transmitter: 35248	Target	Low: 79 mg/dL	High: 140 mg/dL
Sensor: 262344	Alert	Low: 70 mg/dL	High: 200 mg/dL

## Captur AGP in Eversense DMS

The Eversense DMS Dashboard's primary report is the Captur AGP (Ambulatory Glucose Profile) report. The AGP report displays glucose data within 3 sections of their report.

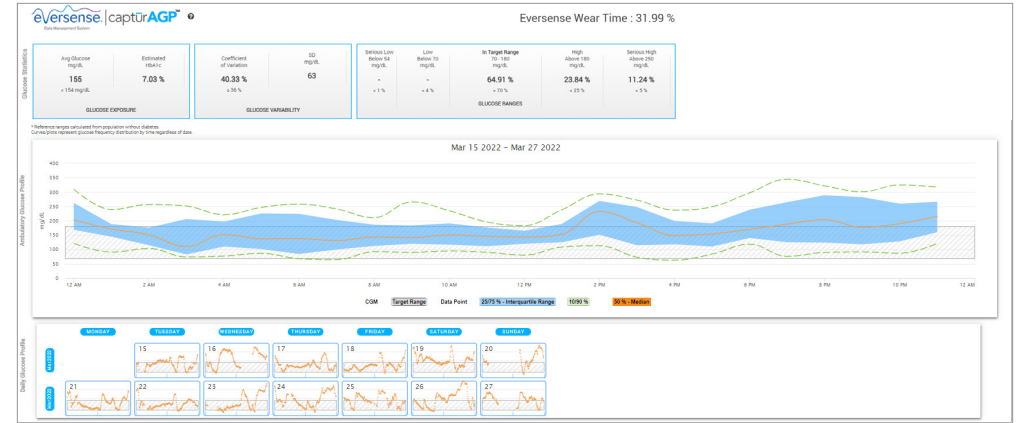



## Glucose Statistics: Metrics about your glucose data



- Avg. Glucose (mean): All glucose values added together, divided by number of readings
- Estimated HbA1c: Calculated from the AGP glucose values; may differ from a lab A1c due to different time periods
- Glucose Ranges: Ideally, glucose values are in target range
  - Low/Very low: Goal – no very low or low values; reducing lows make the patient feel better
  - High/Very High: Goal – no very high or high values; reducing highs will make the patient's A1c lower over time
- SD (Standard Deviation): How much variation is in the data set; ideally a low number
- CV (Coefficient of Variation): How far values are from the average; ideally a low number

**Glucose Profile:** Daily glucose profiles are combined to make a one day (24-hour) picture. Ideally, lines stay within the grey shaded area (target range). Lines may not go all the way across, since the patient may not test every hour of the day.


**Orange:** Median (middle) line where half of the glucose values are above and half are below; ideally, the orange line is mostly flat and inside the grey shaded area

**Blue:** Area between blue lines shows 50% of the glucose values; ideally, space between is narrow

**Green:** 10% of values are above (90% top line) and 10% are below (10% bottom line); ideally, the closer the green lines are to the grey shaded area, the better

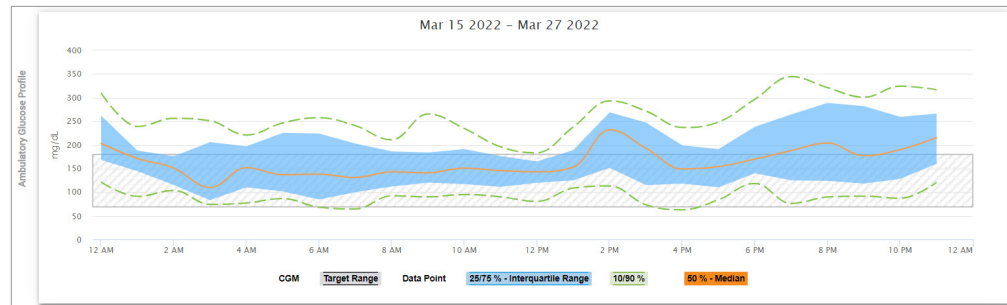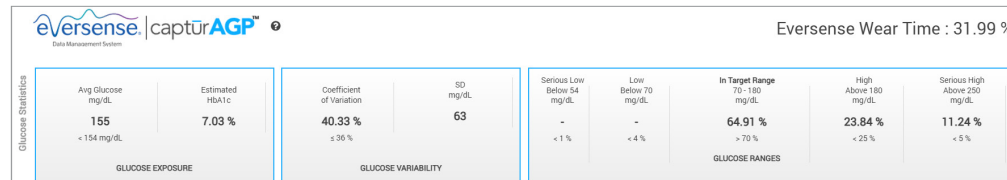

**Daily Glucose Profiles:** Each box is a single day's glucose pattern. It will only display if the date range is set at 14 days or less.

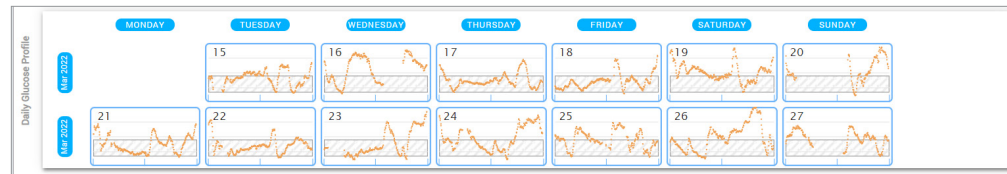

# 5. Reports

Reports provide various ways to organise and visualise glucose readings over time.




**Note:** If you select a report and no data is displayed, check that you have selected a date range in which data is available.

**IMPORTANT:** If autosync is not turned on, data is not being automatically sent to DMS from the Eversense mobile app.

## Print or Email what is displayed in the reports

On the top right of the Home page are options for printing and emailing the Home page reports.



Click			
To	Report Type	Email Report	Save Report as PDF
How	Select which report you want to print or email.	Follow pop-up window prompts for emailing a printer-friendly report to a designated recipient.	Click the icon to convert the report into a PDF that allows you to print the report.

## Report Options

To view the different report options, click on the name of the report in the Report Panel on the Home page.

**Note:** If you select a report and no data is displayed, check the following:

- Expand the date range you have selected. If there is no data over the entire date range over the past 90 days, it may be that the patient does not have autosync turned on in their Eversense CGM mobile app. If autosync is not turned on, ask the patient to turn on autosync.
- Ask the patient to also do a manual upload by tapping “**Start Sync**” in their Eversense CGM mobile app.

5

The screenshot shows a web interface for report selection. At the top, there is a header bar with a blue button labeled "Time In Range" and a "Reports" link on the right. Below this, a large light gray box contains two report options. The first option is "Time In Range" with a date range of "Mar 15 2022 – Mar 27 2022". The second option is "Time in Range by Time of Day" with the same date range and a "6 hours" time interval dropdown menu.

Some reports can be filtered to display certain information. You can select which event types, days of the week, and other options will appear in reports by clicking  next to the report name.

This filter is not available for the AGP, Time in Range by Time of Day, or Time in Target reports.

A blue button labeled "Glucose History" with a small gray funnel icon to its right, indicating a filter option.

## Time in Target


The Time in Target report shows a pie chart that analyses the percentage of time your patient's glucose is below, within, or above the target range the patient sets in the Glucose Settings.

### Viewing the report:

The green wedge represents the percentage of time the patient spent within the target range.

The grey wedge represents the percentage of time the patient spent above the target range.

The red wedge represents the percentage of time the patient spent below the target range.

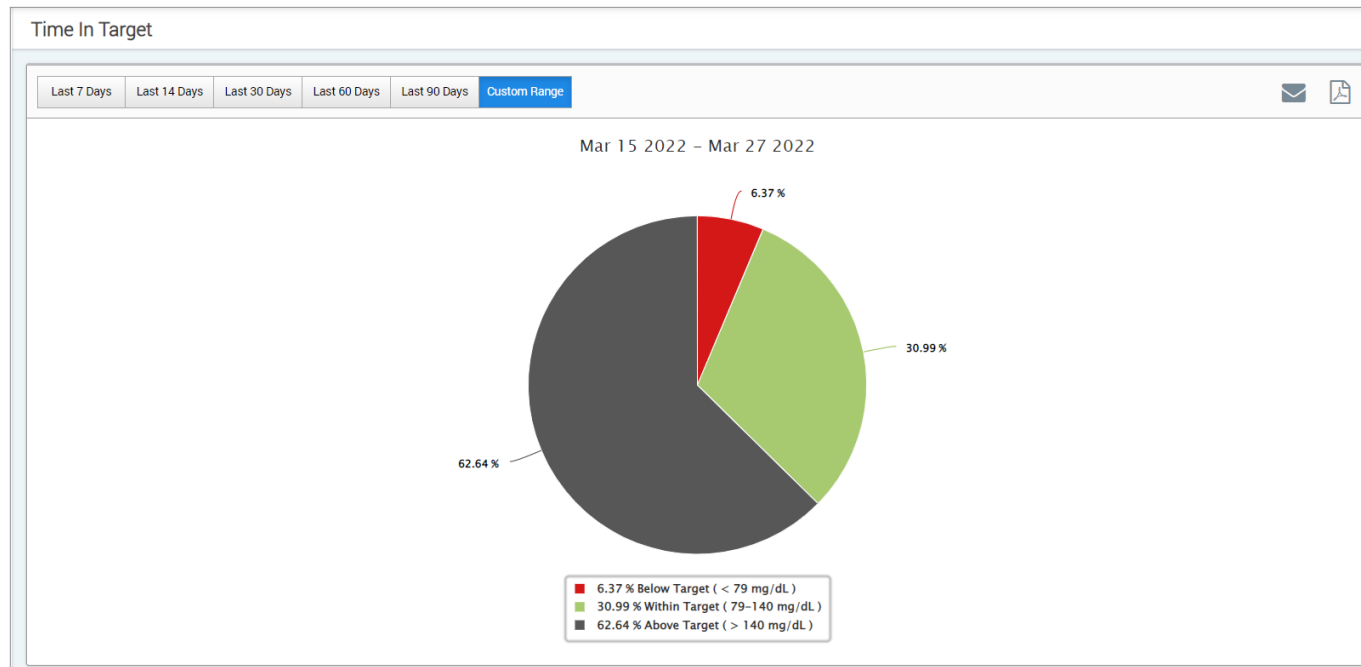

## Time in Range

### Time in Range by Time of Day


The Time in Range by Time of Day report shows a set of bar charts that analyse the percentage of time your patient's glucose is within preset glucose ranges.

- Very Low: < 54 mg/dL or < 3.0 mmol/L
- Low: 54-70 mg/dL or 3.0-3.9 mmol/L
- Within Range: 70-180 mg/dL or 3.9-10.0 mmol/L
- High: 180-250 mg/dL or 10.0-3.9 mmol/L
- Very High: > 250 mg/dL or > 13.9 mmol/L

*There are two sections of this report:*

### Time in Range


This displays your patient's overall percentage of time spent in the preset glucose ranges, based on the date range you have selected.

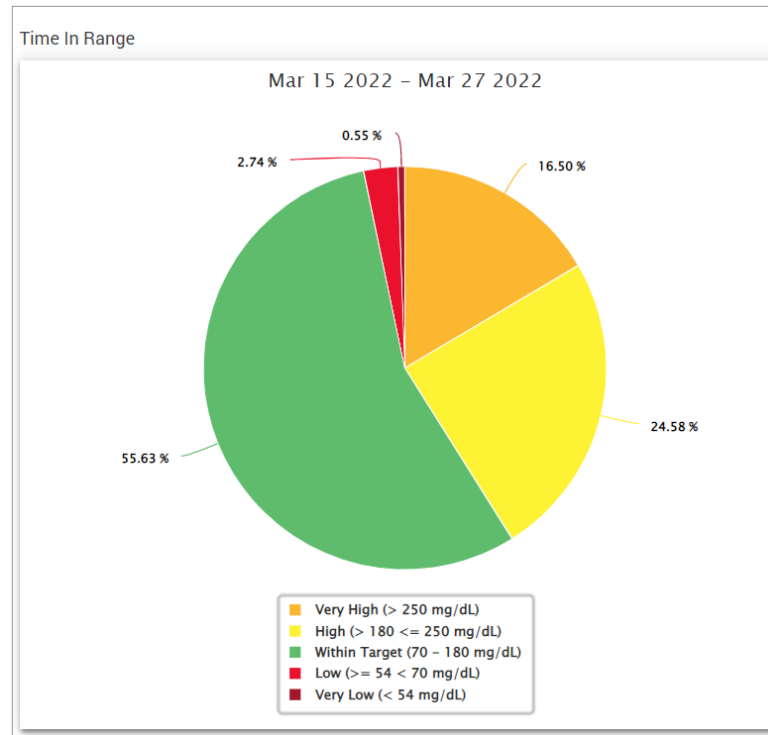

## Time in Range by Time of Day

This section of the report breaks out the overall Time in Range chart into various segmented times of day. You can view your patient's data in segments of 4 hours, 6 hours, or 8 hours. This is adjusted by selecting an option in the top right of the report.

Each segment shows a percentage of time your patient's glucose was within those same segments of Very Low, Low, Within Range, High, and Very High.

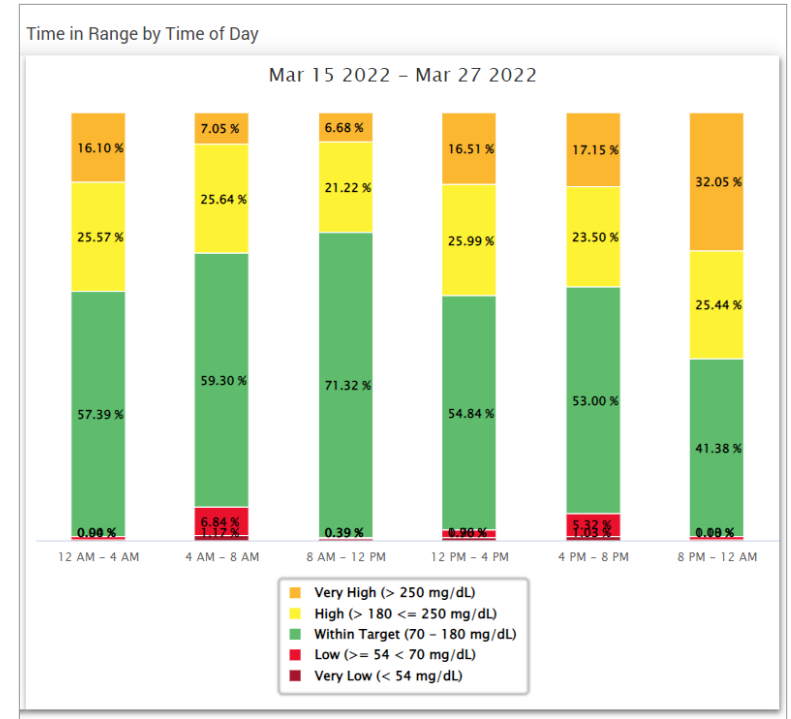



## Glucose Variability Report

*The Glucose Variability Report displays information in two ways:*

### The Trend Graph:

- Displays summary glucose readings over a 24-hour period.
- Includes multiple days of data overlaid on the same graph.
- Shows the maximum, minimum, and average glucose reading for every hour.
- Provides a visual snapshot of variations in your patient's daily glucose levels.

**Note:** Place your cursor above a particular average sensor reading to view details about readings during the hour.

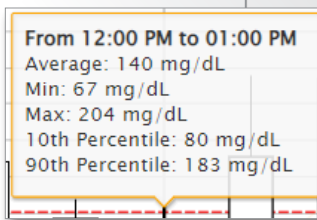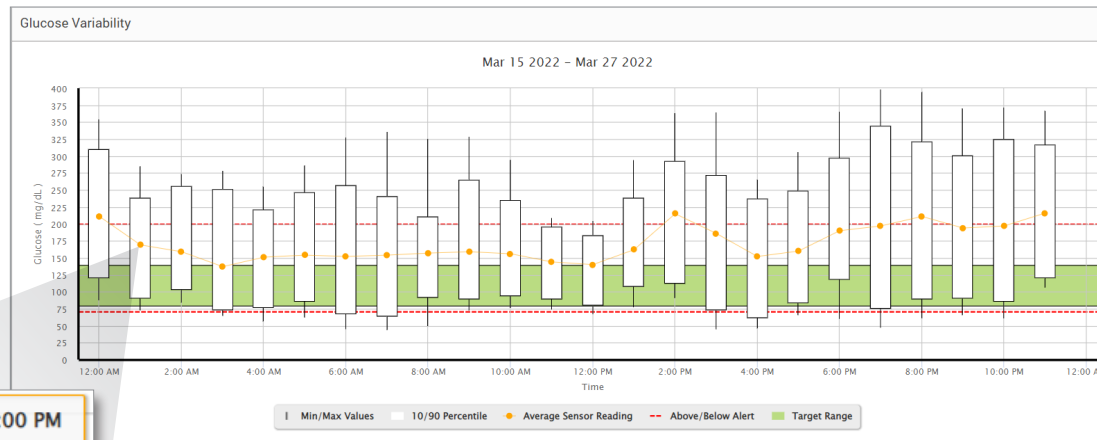



### The Table:

- Organises glucose data by 4 hour time period.
- Provides a detailed statistical analysis for each period.
- Includes the percentage of readings that are within, above, and below the glucose target range.
- Provides a breakdown of insulin usage and carbs entered for those same time periods.

**Note:** If you are having trouble displaying the report, try limiting your date range. Unusually large amounts of data may not allow the report to function.

Glucose			
Glucose	00:00-04:00	04:00-08:00	08:00-12:00
% Below Low Alert	1 %	5 %	-
% Above High Alert	35 %	30 %	17 %
Number of Values Below Low Alert	11	41	4
Number of Values Above High Alert	290	258	145
% Within Target	23 %	28 %	38 %
% Below Target	4 %	8 %	2 %
% Above Target	73 %	64 %	61 %
Average Glucose	182 mg/dL	170 mg/dL	159 mg/dL
Lowest Glucose	49 mg/dL	44 mg/dL	50 mg/dL
Highest Glucose	396 mg/dL	393 mg/dL	329 mg/dL
Standard Deviation	68.3 mg/dL	67.7 mg/dL	53.3 mg/dL
Number of Values in the 10-90 Percentile	673	699	698
Total Number of Glucose Values	838	868	861
Number of Days with at least One Value	21	20	19
Average Number of values per Day	40	43	45

## Glucose Trend Report



### The Glucose Trend Report:

- Displays individual glucose readings over a 24-hour period.
- Displays multiple days of data overlaid on the same graph.
- Provides an easy way to identify variability among the days of the week.

### Viewing the report

Data points in the report are colour-coded with each day of the week displayed in a different colour. You can use the report filter to choose the day/s you want to display on the graph.

**Note:** Place your cursor above a sensor glucose reading to view details about that reading.

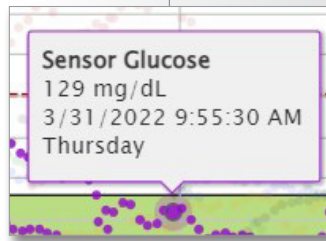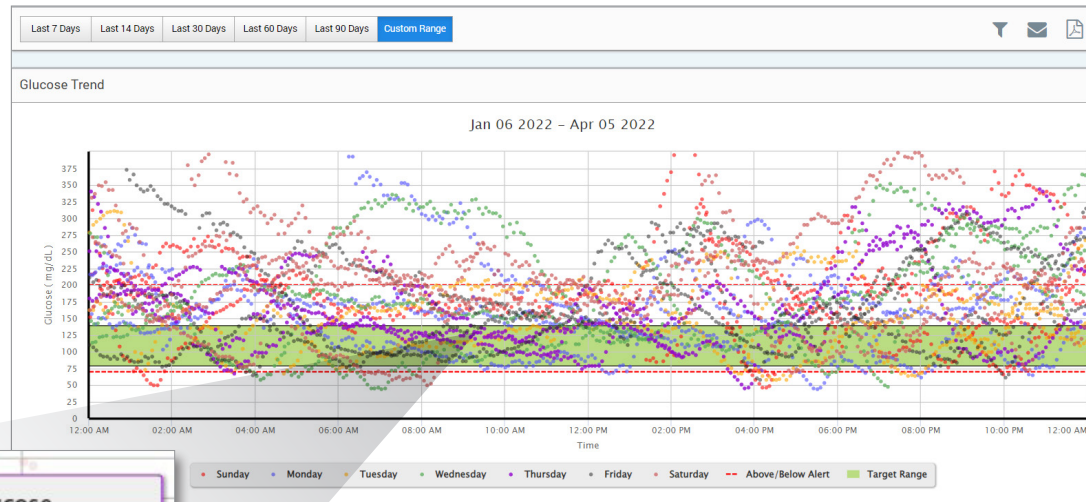


## Glucose History Report

The Glucose History Report displays historical CGM and manually entered data in the form of a trend graph. The Glucose History Report provides an efficient way to track sensor glucose readings and calibrations.

### Viewing the report

Each data point is a sensor glucose reading. The events that were logged by the patient in the mobile app are displayed in the report at the time they were logged. If you select a date range of more than 30 days, events such as meals, insulin, etc. are not displayed.

**Note:** Place your cursor above a sensor glucose reading to view details about that reading. You may also place your cursor above an event icon to view details about that event.

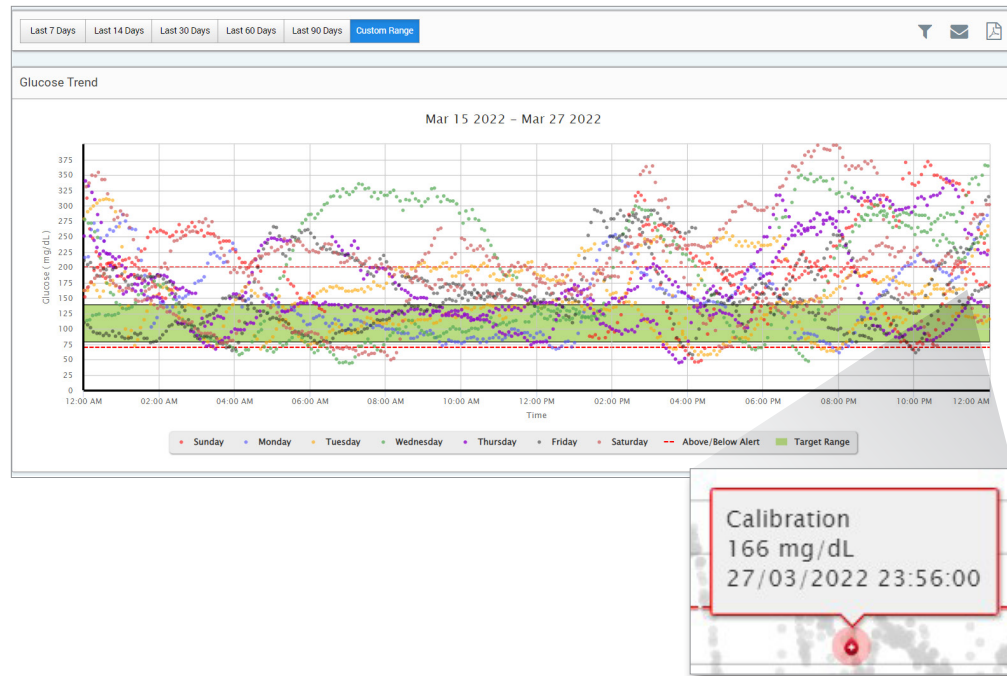

## Daily Glucose Report


The daily glucose report displays glucose readings over a 24-hour period for multiple days in a sequence.

### The Daily Glucose Report:

- Has a fixed glucose target range of 70-180 mg/dL (3.9-10.0 mmol/L).
- Shows high and low glucose alert levels (red lines) as set by the CGM user.
- Shows key statistics for each day based upon glucose target and alert settings.
- Includes an option to display event icons on the trend graph, and to expand a table of events for each day.

### Viewing the report

The Daily Glucose Report displays multiple days in sequence for the date range selected, and provides a way to identify glucose patterns across multiple days. Statistics for each day appear to the right of the trend graph and are based on the standard glucose target range of 70-180 mg/dL (3.9-10.0 mmol/L) and the high and glucose alerts levels set by the patient. To review additional CGM information for each day (alerts, calibrations, events you entered into the CGM app), you can turn on the display of event icons on the trend graph and expand the event table beneath the trend graph and expand the event table to display more information. When expanded, the event table lists information in 4-hour increments aligned with the 4-hour segments in the trend graph.

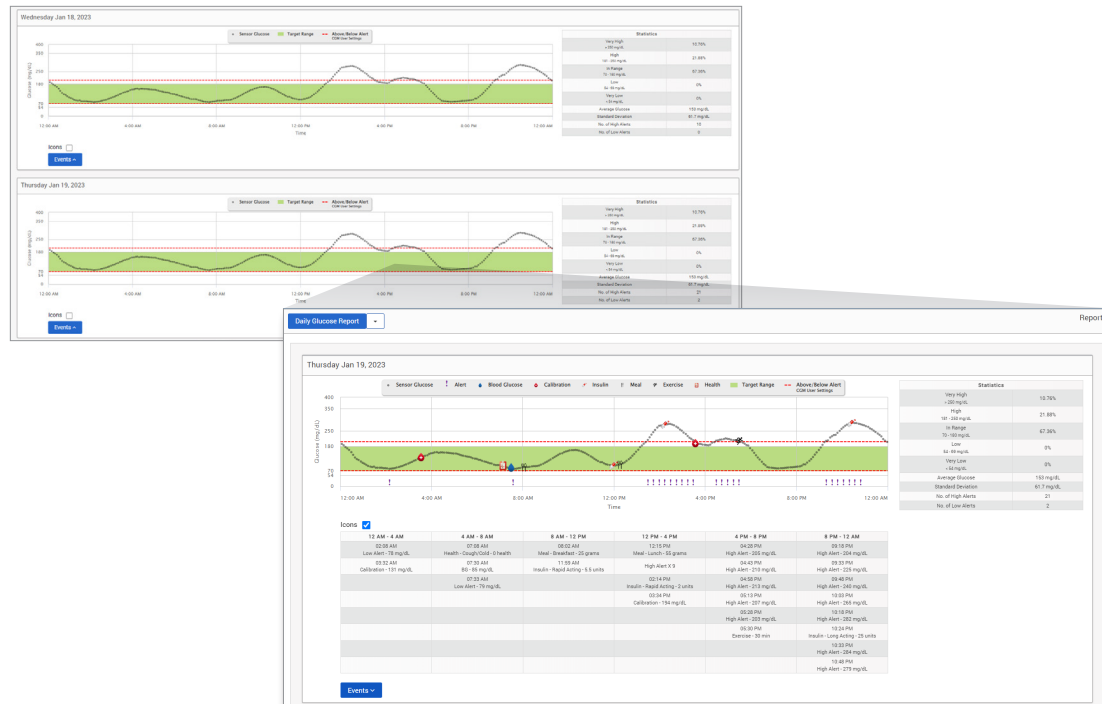

## Statistics:

The statistics table provides analysis of the percentage of time your patient's glucose is within preset glucose ranges.


- Very Low: <54 mg/dL or <3.0 mmol/L
- Low: 54-70 mg/dL or 3.0-3.9 mmol/L
- Within Range: 70-180 mg/dL or 3.9-10.0 mmol/L
- High: 180-250 mg/dL or 10.0-13.9 mmol/L
- Very High: >250 mg/dL or >13.9 mmol/L

Daily average glucose, standard deviation and number of high and low alerts are also included.

## Icons and Events:

The icons and events are displayed based upon your patient's glucose alert settings, calibrations, and any events entered into their CGM app.

**Note:** If desired, you can adjust the CGM alert settings for the report in DMS Pro. (The glucose target settings are fixed at 70-180 mg/dL (3.9-10.0 mmol/L) and cannot be changed.) Adjusting the glucose alert settings will change the placement of the red alert lines on the daily CGM trend graphs, but not the daily statistics. This may assist in decisions regarding the optimal setting for a high or low glucose alert.

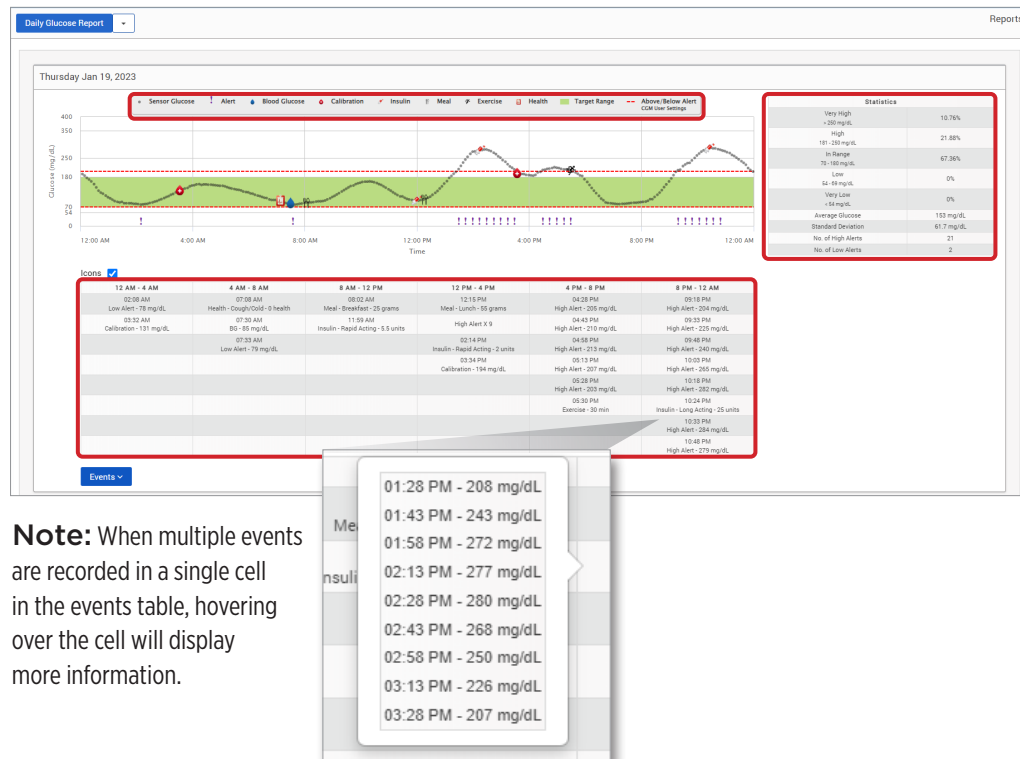

**Note:** When multiple events are recorded in a single cell in the events table, hovering over the cell will display more information.


# Glucose Distribution Report

## The Glucose Distribution Report:

- Organises your patient's glucose data according to their glucose targets and alert ranges and displays the data in pie chart format.
- Displays separate pie charts for the current and previous periods.
- Target ranges are set under Glucose Settings.
- Provides a visual check for the number of readings within target during the predefined range selected and a change over time.

## Viewing the report

The Glucose Distribution Report will display two pie charts based on the predefined date range you select. It will show the current and previous set of data for the date range selected. This data allows you to see a comparison of current to past changes in your patient's management.

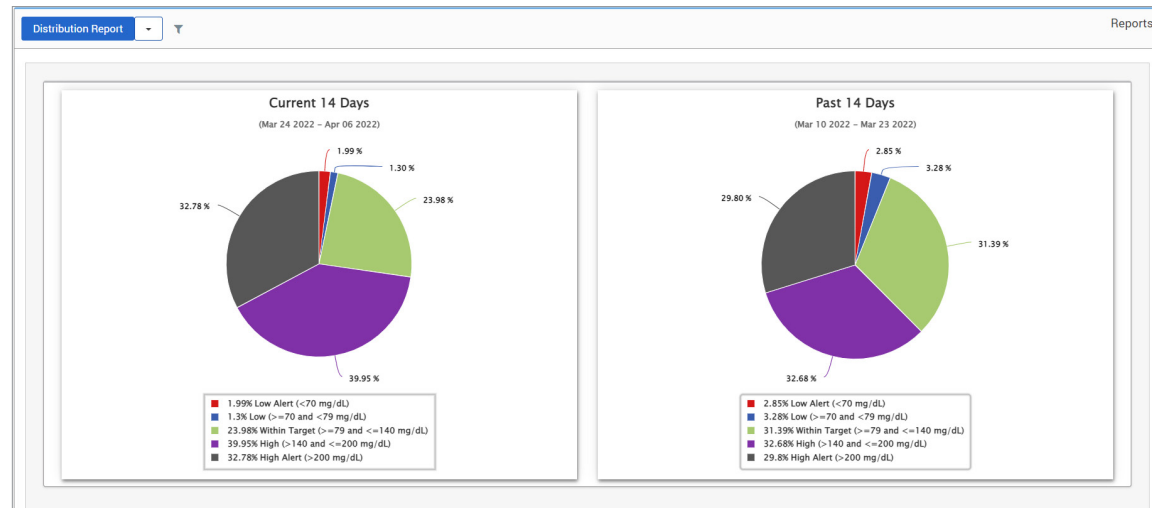

# Transmitter Log Report

The Transmitter Log Report provides a comprehensive data review over the selected date range.

## The Transmitter Log Report displays three separate tables:

- **Blood Glucose:** This displays a list view of all manually entered blood glucose and related events in chronological order.
- **Event Entries:** This displays patient events entered through the Eversense Mobile App.
- **Sensor Glucose:** This displays glucose values picked up by the sensor.

## Viewing the report




Events are displayed with information regarding Date, Time, and:





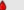
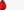




- **Blood Glucose:** Glucose Value, Unit, Calibration
- **Event Entries:** Event Type, Event Sub Type, Value
- **Sensor Glucose:** Glucose Value, Unit, Sensor ID

Events can be sorted in ascending or descending order by clicking the column header.

The number of records is indicated at the bottom of each log.

A symbol under the “**Calibration**” column of the Blood Glucose table will indicate whether a glucose value was  Used for calibration,  Not Used, or  Manual.

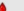-  **Used:** A blood glucose value that your patient entered in the Eversense mobile app as a calibration and is accepted by the system.
-  **Not used:** A calibration that was entered that is over 400 mg/dL (22.2 mmol/L) or below 40 mg/dL (2.2 mmol/L).
-  **Manual:** A blood glucose value that was not entered as a calibration.

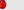Blood Glucose				
<div><input type="text" value="Search"/></div> <div><span>Manual</span> <span>Used</span> <span>Not Used</span></div> <div>Export Data</div>				
Date	Time	Blood Glucose	Unit	Calibration
01-April-2022	11:44 PM	252	mg/dL	
01-April-2022	01:25 PM	223	mg/dL	
01-April-2022	02:24 AM	344	mg/dL	
01-April-2022	01:03 AM	402	mg/dL	
31-March-2022	10:22 AM	103	mg/dL	
30-March-2022	11:31 PM	187	mg/dL	
30-March-2022	02:17 PM	135	mg/dL	
30-March-2022	12:58 AM	153	mg/dL	
29-March-2022	10:22 PM	186	mg/dL	
29-March-2022	07:41 PM	182	mg/dL	
28 Total				<div><div>&lt;&lt;</div><div>&gt;&gt;</div><div>1</div><div>2</div><div>3</div><div>&gt;&gt;&gt;</div></div>

Exporting Data

From the Transmitter Log Report page, you can also export data in a .xlsx format.

- 1. Select **Reports > Transmitter Log**
- 2. Select **“Export Data”** at the top of each table to export the data as an .xlsx (Excel Spreadsheet)

5

Sensor Glucose				
<input type="text" value="Search"/>			<a href="#">Export Data</a>	
Date	Time	Sensor Glucose	Unit	Sensor ID
02-April-2022	09:44	257	mg/dL	262344
02-April-2022	09:39	253	mg/dL	262344






## 6. Settings




### Glucose

Glucose settings allow you to customise the glucose target and alert levels in the reports. This setting impacts how glucose and other event data are organised and displayed in reports. When the patient wirelessly syncs their data via the mobile app, the glucose settings they have saved on the app will also be synced to their DMS account.

**IMPORTANT: Glucose settings DO NOT get synced when data is uploaded from the smart transmitter via the USB Cable.**

Patient Name: Jean Garcia Age: 21 Gender: Female MRN:  jeangarc97@gmail.com  Jon Johnson, MD 

Last 7 Days Last 14 Days Last 30 Days Last 60 Days Last 90 Days Custom Range

Glucose Settings

To change the settings in the Eversense DMS Pro view, follow the steps below:

1. Click Glucose Levels near the top of the screen

2. Click the mg/dL or mmol/L button below the Units Of Measure to select the desired unit of measure for displaying glucose data.

If you change the unit of measure, the glucose range and alert range values will change accordingly.

3. Click the +/- buttons to the right of the High Target and Low Target to display and adjust the high and low limits of your glucose Target Range

4. Repeat for the glucose Alert Range

5. When you have finished making changes, click “Save” to save your changes

**Note:** These settings will be saved for the next time you view this patient’s reports, even after logging out of DMS Pro. Eversense DMS Pro settings do not affect the patients’ Eversense CGM System settings or the Eversense DMS display.


The screenshot shows a 'Glucose Settings' window. At the top, it says 'Glucose Settings'. Below that is a section for 'Unit Of Measure' with two radio buttons: 'mg/dL' (selected) and 'mmol/L'. Underneath are two columns of settings. The first column is 'Target Range (must be between Alert Settings)' with 'High Target \*' set to 140 and 'Low Target\*' set to 79. The second column is 'Alert Range (must be between 60 and 350 )' with 'High Alert\*' set to 200 and 'Low Alert \*' set to 70. Each value has minus and plus buttons for adjustment. At the bottom are 'Cancel' and 'Save' buttons.

# 7. Help

## About

This page shows the most up-to-date versions of the Eversense DMS Pro.

**IMPORTANT:** For questions about your Eversense DMS Pro account, contact your local distributor. In the U.S. call 844-SENSE4U (844-736-7348).

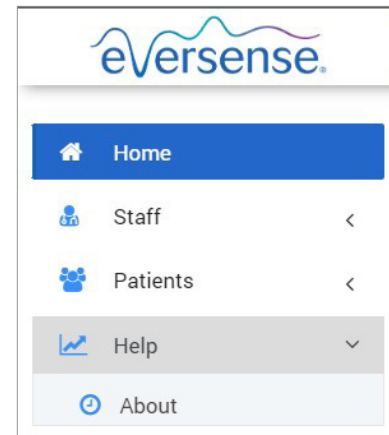

## 8. Troubleshooting

---

1. **My previous filters are no longer applied upon going back to my Home page or Patients page.**
  - Filters are refreshed upon leaving the Home page or Patients tab. Please reapply filters to see the desired list of patients.
2. **If the only clinic admin inadvertently removes their admin privileges please contact Customer Care.**
3. **CGM users in Germany may decline to create an Eversense account. These users cannot share data to Eversense DMS Pro.**

# Notes

---

---

---

---

---

---

---

---

# Notes

---

---

---

---

---

---

---

---



Basic UDI/DI:

- Eversense DMS Pro: 081749102FG5800U5

Call 04-6175390 for Customer Support

Distributed by:

DYN Diagnostics Ltd.

7 Ha'eshel St. P.O. Box 3063

Caesarea Industrial Park 3079504, Israel

Phone: 04-6175390

Email: [patient.care@dyn.co.il](mailto:patient.care@dyn.co.il)



**Senseonics, Inc.**

20451 Seneca Meadows Parkway  
Germantown, MD 20876-7005 USA

[global.eversensed diabetes.com](http://global.eversensed diabetes.com)

Patents:

[www.senseonics.com/products/patents](http://www.senseonics.com/products/patents)



**Emergo Europe**

Westervoortsedijk 60

6827 AT Arnhem

The Netherlands



(241) LBL-4207-31-201\_REV\_C



Dimensions	
Trim: 11.75" w X 5" h	Bleed: 0.125"
Folded <i>(Include folded dimensions if applicable):</i>	

Colors: 4/color
Spot <i>(Name PMS colors if applicable):</i> N/A

Paper Weight:
Cover: 65# Gloss Cover
Text: 60# Gloss Text